

Bioline™ CHOLERA Ag O1/O139

V. CHOLERAЕ O1/ O139 ANTIGEN TEST

Bioline™ Cholera Ag O1/O139 test is a rapid immunochromatographic assay for qualitative detection of *Vibrio cholera* O1/O139 in human fecal specimens.

- Easy to use, convenient and clean test
- Specimen : Solid fecal specimen (about 50 mg) or liquid fecal specimen (300 µl)
- Test result : 15 minutes
- Shelf life and storage temperature: 24 months from the date of manufacturing at 1-30 °C
- Performance: Sensitivity 100 %, Specificity 100 % (vs. Culture)



MATERIALS PROVIDED

- Test device
- Specimen collection tube with extraction buffer
- Specimen collection swab for solid fecal specimens
- Specimen collection dropper for liquid fecal specimens
- Patient identification label

SIMPLE PROCEDURE

- 1 Open the cap of the specimen collection tube**
- 2 Insert swab specimen**
Insert the swab specimen or transfer liquid specimen.
- 3 Assemble dropping cap and open the nozzle cap**
- 4 Add Specimen**
Dispense 3 drops.

RESULTS INTERPRETATION

Wait 15 mins.

15 MIN

The presence of any test line, no matter how faint, the result is considered positive.

ORDERING INFORMATION

| PRODUCT | CAT. NO. | TYPE | SPECIMEN | PACK SIZE |
|--------------------|----------|--------|----------|-----------|
| Cholera Ag O1/O139 | 44FK30 | Device | Fecal | 20T/Kit |